MRSA-Anreicherungs-Bouillon

Bouillon d'enrichissement SARM

MRSA Enrichment Broth Base

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Bibliographie

1. ASM Approved Guidelines – 1996 : Guidelines for Assuring of Medical Microbiological Culture Media
2. C. Cuny et al.: Diagnostics of staphylococci with special reference to MRSA; J. Lab. Med. **26**, 165–193 (2002).
3. Derek F. J. Brown et al.: Guidelines for the laboratory diagnosis and susceptibility testing of methicillin-resistant Staphylococcus aureus (MRSA) J. Antimicrob. Chemother. (December 2005) 56 (6): 1000-1018

Application

The enrichment broth base serves for selective enrichment and screening of methicillin-resistant *Staphylococcus aureus* (MRSA, synonymous with oxacillin-resistant *Staphylococcus aureus*) from clinical specimens. The MRSA enrichment broth base may also be used as a transport medium for specimens.

Package content

100 tubes filled with 3.0 ml of MRSA enrichment broth base.

Test principle

The MRSA enrichment broth base contains certain nutrients and growth factors which facilitate rapid growth of MRSA as well as additives which inhibit the growth of other bacteria.

Typical composition

MRSA enrichment broth base (yellowish, clear low-viscosity liquid)
brain heart infusion (from fresh tissue)
peptone
sodium chloride
gelatine
dextrose
disodium phosphate
inhibiting agents
pH 7.2

Special warnings and precautions for use

Do not use the enrichment broth if you observe the following:

- discolouration or turbidity of the culture medium
- leakage of the culture medium
- signs of bacterial or fungal growth

Avoid contact with the growing colonies as they may have pathogenic properties.

Storage

Store the enrichment broth at a temperature of +4 to +8°C. Do not freeze. Do not use the product after the expiry date indicated on the packaging (EXP).

Sampling

To avoid contamination, make sure the growth medium only gets in contact with the swab or appropriate testing material, but with no other materials. It is important, though, that the growth medium has full contact with the material to be tested. Screw-lock the tube tightly after sampling.

Execution of the test

1. Take the sample by using a swab or other appropriate testing material.
2. Inoculation by immersion
Dip the swab or testing material into the liquid for three to four seconds while slightly rotating the swab. Nearly all of the sample should be absorbed.
3. Screw on the lid tightly.
4. Write the variable data onto the label.
5. Keep the container tightly closed and incubate the broth base for one day at a temperature of 35 to 37°C. If no growth is visible after this incubation period, you may incubate for another 24 hours.

Transport of inoculated MRSA enrichment broth base

The inoculated enrichment broth can be transported at temperatures between 15 and 20°C.

Interpretation of results

If incubation of the MRSA enrichment broth base results in visible turbidity, this may be an indication of methicillin-resistant *Staphylococcus aureus* (MRSA) or *Staphylococcus aureus*.

A subculture from the MRSA selective medium with additional methods of differentiation is necessary.

Methicillin-resistant *Staphylococcus aureus* grows in the enrichment broth. The growth of nearly all other bacteria, yeasts and moulds is inhibited by the additives.

The presence of MRSA is always an indication of a lack of cleanliness on the part of the patient or staff and their environment.

Quality control

The following test strains can be used for quality control:

<i>Escherichia coli</i> ATCC 25922	–
<i>Staphylococcus aureus</i> ATCC 25923	+
<i>Staphylococcus aureus</i> ATCC 33591	+

(+) growth (–) partial or complete inhibition

Disposal

It is the responsibility of every laboratory to dispose of the arising waste materials safely and appropriately in accordance with the legal requirements.

Literature

1. ASM Approved Guidelines – 1996: Guidelines for Assuring of Medical Microbiological Culture Media
2. C. Cuny et al.: Diagnostics of staphylococci with special reference to MRSA; J. Lab. Med. **26**, 165–193 (2002).
3. Derek F. J. Brown et al.: Guidelines for the laboratory diagnosis and susceptibility testing of methicillin-resistant *Staphylococcus aureus* (MRSA) J. Antimicrob. Chemother. (December 2005) 56 (6): 1000-1018

**Abkürzungen/Abréviations/Abbreviations
Symbole/Symboles/Symbols**

Symbol/Symbole	Bedeutung/Signification
	Bestellnummer/Numéro d'article/Catalogue number
	Packungsinhalt/Contenu du coffret/Contents of kit
	In vitro diagnosticum/Usage in vitro/ For in vitro diagnostic use
	Hersteller/Fabricant /Manufacturer
	Lagertemperatur/Température de conservation/ Storage temperature
	Verwendbar bis/Date de péremption/Use before
	Chargenbezeichnung / Numéro du lot/Lot number
	Gebrauchsanweisung beachten/ Consulter le mode d'emploi/ Consult the instructions for use
	Einmalgebrauch/ Usage unique/ single use
	CE-Kennzeichnung/CE marquage/CE marking